

**Kelapher**<sup>®</sup>  
Cream Patented

Medical device for topical use

EXTERNAL USE  
50ml e<sub>tube</sub>

**TDC**

Our skin is the first barrier between the inside and outside of our body against harmful or potentially toxic elements such as heavy metals. In fact, under certain circumstances, especially iron, lead and zinc, accumulating on the derma can trigger a series of reactive chains which result in the production of free radicals. In these cases, the Kelapher Lactoferrin is able to fully exert its chelating activity against the metal ions through its embedding into liposomes (Lipid Technology), for the treatment of all trofic-dyschromic alterations of the dermal-epidermal tissue (European patent). Liposomes are nothing but lipid particles with a structure similar to that of cell membranes. The advantage of this technology is not only to obtain an increased bioavailability, but also enhance the protection against Lactoferrin denaturation.

**Packaging:** 50 ml aluminum tube in a cardboard box.

**Composition:** water, Lactoferrin LT (Lipid Technology), propanediol, sepigel 305, myristyl myristate, shea butter, ethylexyl palmitate, jojoba oil, dimethyl sulfone, acrylates/C10-30 alkyl acrylate crosspolymer, carbomer, disodium EDTA, sodium hydroxide, ethylhexyl glicerina, caprylyl glycol, phenoxyetanol.

**Indications:** Kelapher cream thanks to its antioxidant properties associated with a chelating activity against metallic ions is useful in dyschromia resulting from ematomas, ecchymosis, post-operative ecchymosis, Post sclerotherapy haemosinergic dyschromia, CVI (Chronic Venous Insufficiency) dyschromia and phlebostatic ulcers, as well as in non-ablative post-laser treatments, epilation and vascular lasers.

**Instructions for use:** Apply twice a day until the symptoms disappear.

**Warning:** Keep out of reach of children. Do not swallow. Do not use the product if the packaging does not appear intact. Do not use after the expiry date. The expiry date refers to the product is intact and properly stored. The use, especially if prolonged could lead to sensitization. Should this occur, discontinue the treatment and consult a physician in order to follow an appropriate therapy. It is important to inform your doctor or pharmacist of any side effects not listed in this leaflet.

**Precautions for use:** In pregnant and lactating women and in children (under 3 years), it is advisable to consult a doctor before the use. Do not use the product on open wounds. The simultaneous use of creams and / or similar on the same anatomical part may preclude the efficacy and the safety of the device.

**Contraindications:** Known hypersensitivity to components.

**Storage:** Store the package at temperature not exceeding 25°C, in a dry place, away from light and heat sources.

**Distributor - Greece, Cyprus:**

Meditrina Ltd., Irakleitou 117, 15238 Halandri, Tel. +30 210 6726260 email:info@meditrina.gr

**European Patent**

Manufacturer:

TDC Technology Dedicated to Care S.r.l. - Via Copernico, 38 - 20125 Milano (Italy)

CE 0425

[REF] ES00587TDC A1 rev. 1 of the 03/12/2019